[www.arcmicrotech.com](http://www.arcmicrotech.com)  
[www.arcequine.com](http://www.arcequine.com)

**ARC Microtech Limited**  
Unit 7, Woodland Enterprise Centre, Hastings Road  
Flimwell, East Sussex TN5 7PR, UK

© 2021 ARC Microtech Limited  
Patent Numbers GB 2502133 & GB 2534681  
Ver. 6.1

## User Manual

Supports and aids the body's  
own natural healing process

## What is the ArcEquine Device?

ArcEquine is a remarkably small Device that is wearable, drug free and non-invasive. It works by automatically generating a complex series of microcurrents with a simple, one-finger operation. In fact, it delivers over 4.6 million permutations within a single programme to support the body's own natural repair process. And just like naturally occurring microcurrents, they can't be felt, so cause no discomfort to the horse at all. It is supplied subject to our terms and conditions of purchase available on our website. Please ensure that you read these prior to using the Device.

## Answers to most frequently asked questions

- No, the Device cannot be placed over an open wound or skin irritation.
- Yes, the ArcEquine is safe to use on injuries that have not broken the skin, providing the horse is accepting of this.
- Always rotate the Device between legs for each 3-hour delivery cycle.
- Like any electrical device, do not use in wet or damp conditions. Dry wet legs prior to use.
- Never allow anyone to operate the Device without reading this User Manual. Please use the Device only as directed.

## What's in the box?

- The Device
- Leg Strap
- Leg Strap Liner
- Carbon Rubber Pads (x2)
- Conductive Gel (250ml)
- Charger
- User Manual


## Programmes

We always recommend you run through the following 6-week protocol first in order to maximise results. Once this initial period finishes you should use programme 4 (**P4**) for a minimum of two days per month, or you can select the individual programme that best suits your horse's condition. If there is an injury, use the Device every day on the most appropriate programme. Every few months, we recommend running through the 6-week protocol again in order to maintain optimum results. Please note that this is the manufacturers guidance and not a clinical recommendation.

 Indicator lights

 On/Off and Programme button

 Play button

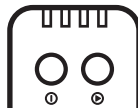## 6 Week Protocol:

**P1** = Pain 75% / tissue repair 25%.  
Daily use for **1 week**.

**P2** = Pain 50% / tissue repair 50%.  
Daily use for **2 weeks**.

**P3** = Pain 25% / tissue repair 75%.  
Daily use for **3 weeks**.

**P4** = Pain 0% / tissue repair 100%.  
Maintenance programme.

**Each programme lasts for 3-hours.**

## Programming the Device

- On/Off button. Press and hold for 3 seconds when turning on or off.
- All four indicator lights will very quickly show green, then red to indicate that your Device is functioning correctly.
- After this, lights will flash green corresponding with the battery charge: 1 light = 25%, 2 = 50%, 3 = 75%, 4 = 100%.

- After 3 seconds all lights will go out. Immediately after this, a red light will indicate which programme was last used.  
1st light = **P1**, 2nd = **P2**, 3rd = **P3**, 4th = **P4**.
- Press the left (On/Off) button quickly to skip through and select the desired programme from **P1** to **P4**.
- Press and release the right play button quickly to activate the selected programme. The unit will beep on activation of the selected programme.
- Once activated, the right play button will turn either green (connected) or red (not connected). After 30 seconds the light will go out to conserve battery, but the programme indicator will continue to flash throughout the delivery cycle. **If not connected, ensure you have used sufficient conductive gel on the carbon rubber pads.**
- Once the 3-hour delivery cycle is complete, the Device will turn itself off.

## How to Put the Device on

Only apply the Device to unbroken, clean and healthy skin.

**Leg Strap Liner:** Align the two holes with the two press studs on the leg strap.

**Carbon Rubber Pads:** Fix to the leg strap using the press studs. This will also secure the leg strap liner. Carbon rubber pads can be wiped clean with warm water after each use.

**Conductive Gel:** Apply a small amount to the carbon rubber pads. The gel ensures a good electrical connection with the skin and is Propylene Glycol free.

**Please Note:** Do not use any other gel in conjunction with the Device.

**Leg Strap:** Attach the leg strap with the Device to the outside of the cannon bone. Rotate the strap between different healthy legs each day. The carbon rubber pads are very sensitive and only light pressure is required to maintain connectivity. After putting the Device into the leg straps elasticated bands, please ensure that the electrical lead in the leg strap is inserted into the bottom of the Device. Do not over tighten and protect with a soft bandage.

- Never strap the device over an open wound or skin irritation.
- The ArcEquine is safe to use on injuries that have not broken the skin, providing the horse is accepting of this.

- Always rotate the device between legs for each 3 hour programme.

## Sharing your Device

If you intend to share your Device with another, please ensure this is only after appropriate inspection and thorough cleaning with medical disinfectant wipes (such as AzoMax). The accessories should not be shared, with each individual using or purchasing their own.

## Stables

If others are looking after your horse, please ensure that they read this User Manual before using the Device. Do not instruct them on how to use it yourself.

## Charging

Please inspect the state of the plug charger (DC PSU) before use and blow out debris if required.

Only use the charger supplied, or an original replacement charger ordered directly from Device Manufacturer.

On receiving the Device, please charge fully before initial use. One 4-5 hour charge can give up to 30 hours of use, enough for 10 delivery cycles.

Insert the charger lead into the base of the Device. Plug into a power socket. Two beeps indicate charging.

Battery charge is indicated by the corresponding number of flashing green 'indicator lights':  
1 light = 25%, 2 = 50%, 3 = 75%, 4 = 100% battery charge.

All indicator lights off = 100% charged.

**Please note:** The ArcEquine uses a lithium-ion battery, all ARC delivery units need to be charged on a reasonably regular basis in order to maintain optimal battery life and condition. Avoid long periods without using or recharging the Device.

## Turning the audible beep on and off

Press and hold both buttons at the same time for 3 seconds. Three short beeps will play:

- High pitch – High pitch – High pitch = audible beep ON.
- High pitch – High pitch – Low pitch = audible beep OFF.

## Important

- Do not place or strap the Device over an open wound or skin irritation.
- Do not apply any other creams or gels to the carbon rubber pads other than ARC approved conductive gel.
- The Device is in addition to, and in no way a substitute for, your existing veterinary policy and you should always consult a veterinary expert on equine conditions prior to using the Device.
- Like any electrical device, do not use the Device in wet or damp conditions.

- Some horses may, after prolonged use, develop a sensitivity reaction. This can manifest in a 'rub' type condition and even localised hair loss or discolouration. \*We recommend that the leg being used is alternated after every 3-hour cycle.
- If this occurs, users should stop using the Device and should contact the Device Manufacturers if the horse experiences any adverse reactions. Our contact details are on the back page.
- Please use the Device only as directed.

## General Warnings

- Do not immerse any part of the unit in water.
- Use this Device only with the leads, electrodes and accessories recommended by the manufacturer. Use of other parts or materials not supplied by the manufacturer will invalidate the warranty.
- Keep the Device away from fireplaces or radiant heaters as the heat may affect the Device.
- Use of microcurrent Devices may cause some dehydration. Please ensure your animal always has access to fresh, clean water when using the Device.

## Troubleshooting

- Right play button showing red = loss of connectivity. Please refer to the Good/bad connectivity notes below.
- Left On/Off button showing red = lack of charge. Device unit needs charging. Please refer to the Charging notes above.
- Device keeps switching off = the right play button was not pressed.
- Audible beep has gone = Device's default setting is to have audible sounds. Please refer to notes for turning the audible beep on and off.

## Good/bad connectivity

If the right play button turns green, the programme has started and you have good connectivity.

If the right play button turns red, the programme has started but you DO NOT have connectivity.

If RED, please check that:

- the electrical lead from the strap is still attached to the bottom of the Device;
- the carbon rubber pads have not come loose or fallen off;
- the carbon rubber pads have sufficient gel applied.

## Warranty

Our limited warranty covers defects in materials and workmanship or electrical fault under normal use for 2 years. This is valid from the date of delivery of the Device to the original purchaser. If a defect arises and a valid claim is received within the warranty period, we will either refund the cost of the defective Device or exchange the Device with a Device that is new or which has been manufactured from new or serviceable used parts.

**Exclusions:** Carbon Rubber Pads, Conductive Gel, Leg Straps, Leg Strap Liners and Chargers are not covered by this warranty - only the **Device**.

**This warranty is set out in and subject to our terms and conditions of purchase.**

## WEEE Statement

The Symbol on the product indicates that it should not be disposed of with your normal household waste but should be taken to a designated collection point for the recycling of waste electrical and electronic equipment.

## RoHS statement

The Device is subject to, and compliant with, the requirements of the Restriction on Hazardous Substances (RoHS) Directive, 2011/65/EU.

## Standard Symbols

Manufacturer

## Contact details

For further information please direct any general enquiries to:

admin@arcmicrotech.com

Tel: +44 (0) 1580 755504

**Please note:** All enquiries will be handled between 09:00 – 17:00 Monday - Thursday, 09:00 – 16:00 Friday.

Please visit [www.arcequine.com](http://www.arcequine.com) or [www.arcmicrotech.com](http://www.arcmicrotech.com) should you need more information on our Device (including our FAQs) or if you need to buy more consumables.

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